


# TWO-HANDED (WINDLASS) TOURNIQUET APPLICATION


## TACTICAL FIELD CARE (TFC)



**EXPOSE** and **ASSESS** the wound.

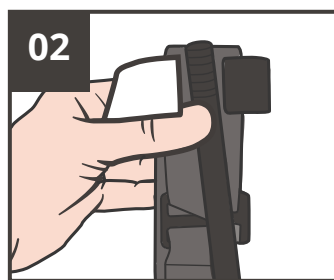


**REMOVE** tourniquet from the casualty's JFAK and/or carrying pouch.

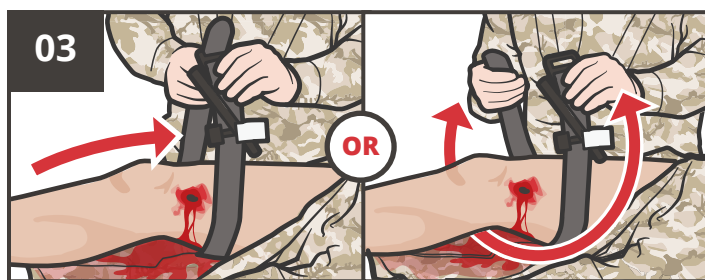


**INSERT** wounded extremity through the loop of band (looped).

**ROUTE** band around limb, and through routing buckle.

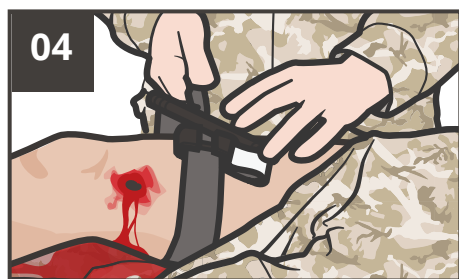


**POSITION** 2-3 inches above wound directly on skin.

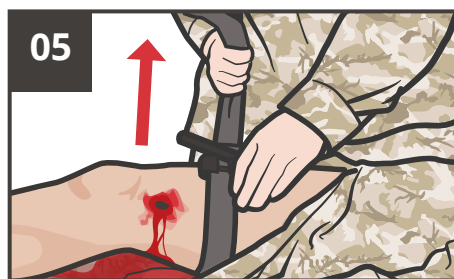


**PULL** self-adhering band as **TIGHTLY** as possible.



**FASTEN** it back on itself all the way around the limb.

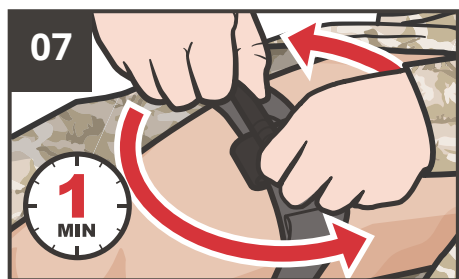


**TWIST** the windlass rod until bleeding has stopped (*complete steps 1-6 in under 1 min*).

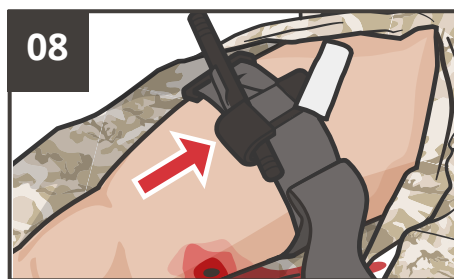


**LOCK** the windlass rod in place with the windlass clip.




**ROUTE** self-adhering band around the rod and between the clips.

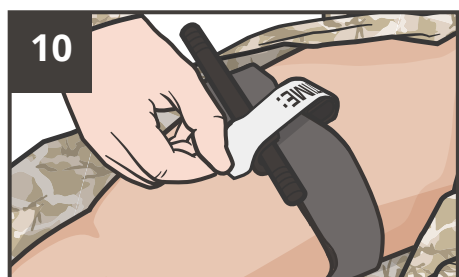

**SECURE** with the windlass safety strap.

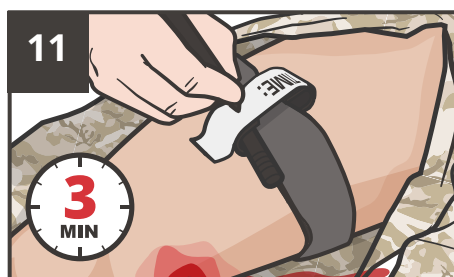

**ANNOTATE** time of tourniquet application.

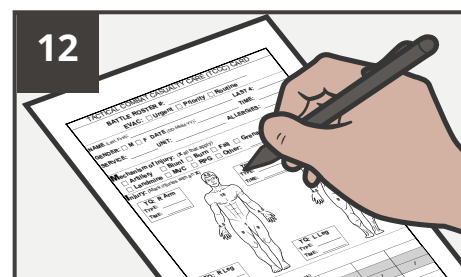

**DOCUMENT** all findings and treatments on the DD Form 1380 TCCC Casualty Card and attach to casualty.